


DIRECTIONS FOR USE: STRETCH-A-TRAY™

Reorder No. 1147

1. PLACE ON MAYO STAND, BACK TABLE STAND, BACK TABLE OR PATIENT.
O P T I O N A L: PEEL ADHESIVE TAB AT BOTTOM OF TRAY AND ADHERE
IN DESIRED POSITION.
2. ADJUST TRAY TO DESIRED LENGTH.
3. PLACE SHARP ITEMS IN TRAY WHEN PASSING.

CAUTION: ENSURE SHARP INSTRUMENTS ARE PROPERLY POSITIONED
INSIDE THE TRAY TO AVOID POTENTIAL INJURIES.

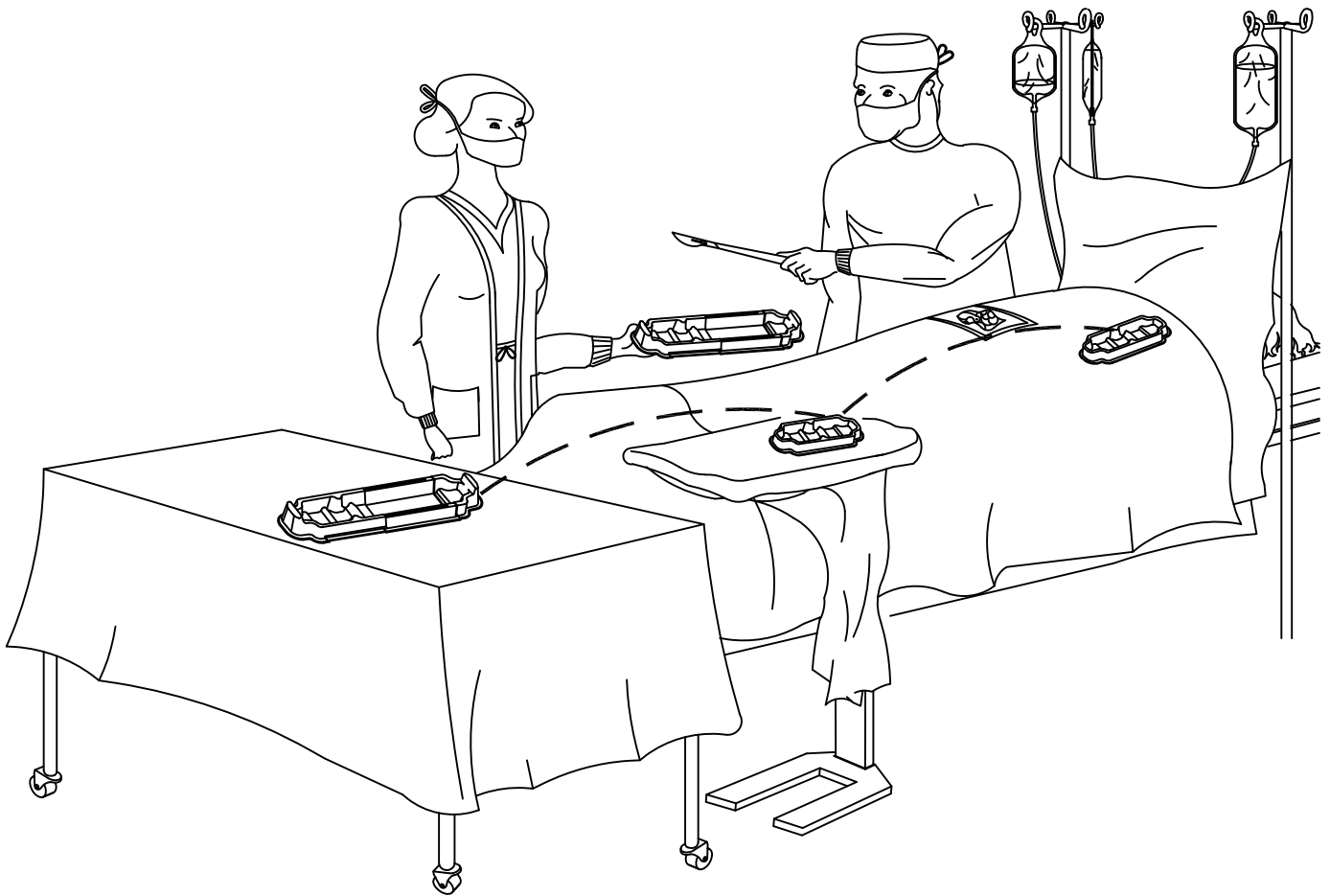


SANDEL LLC
Leaders in Healthcare Safety

Sandel Medical Industries, LLC
19736 Dearborn Street, Chatsworth, CA 91311
1-818-534-2500 www.sandelmedical.com

Safety means NEVER having to say you're sorry.®

In the healthcare industry, the color orange is a trademark of Sandel Medical Industries, LLC