


DIRECTIONS FOR USE: Change-A-Blade™ Safety Scalpel Handle


Pat. # US D568,475 S; D542,415

Reorder No. 2200


1. REMOVE TIME OUT® SLEEVE PRIOR TO USE.

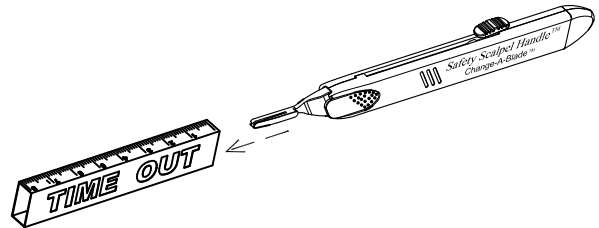


2. TO DISENGAGE SAFETY SHIELD, PRESS DOWN THE BUTTON AND SLIDE BACK.

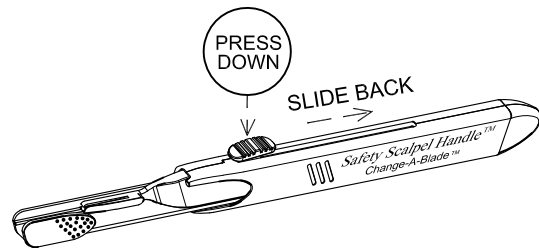

3. SAFELY LOAD BLADE ONTO SCALPEL.
(USE THE CHANGE-A-BLADE WITH AS MANY BLADES AS REQUIRED FOR THE SURGICAL PROCEDURE)

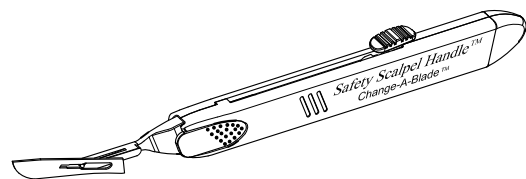

4. AFTER THE BLADE IS SECURELY LOADED, SLIDE THE BUTTON FORWARD UNTIL THE SAFETY SHIELD IS LOCKED IN TO PLACE.

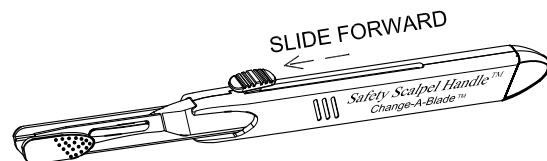

5. TO REMOVE THE BLADE, USE SANDEL'S SCALPEL DISARMER™ (REORDER #2241)

NOTE: ACCOMMODATES BLADE # 10, 11, 15, & 15C
CAN NOT BE USED WITH BLADE #12 OR 12B.



Sandel Medical Industries, LLC
19736 Dearborn Street, Chatsworth, CA 91311
1-818-534-2500 www.sandelmedical.com