


DIRECTIONS FOR USE:

CORRECT SPECIMEN ZONE™ - ORANGE


Pat. # US 7,381,288

Reorder No. 3450

- 1 PLACE THE SPECIMEN STRIP AND THE PERMANENT MARKER ON THE BACK TABLE.

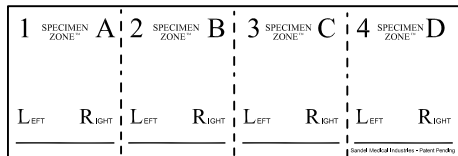


- 2 PLACE THE SPECIMEN ON THE SPECIMEN PAD.
WRITE THE SPECIMEN NAME ON THE PAD.
Optional: Identify left & right
WHEN READY TO TRANSFER TO THE CIRCULATING NURSE, TEAR OFF THE PAD SECTION WITH THE SPECIMEN.

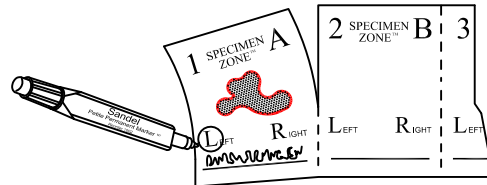

- 3 PASS THE SPECIMEN PAD TO THE CIRCULATING NURSE.



- 4 TRANSFER THE SPECIMEN INTO THE APPROPRIATE CONTAINER FOR TRANSPORTING AND LABELING.

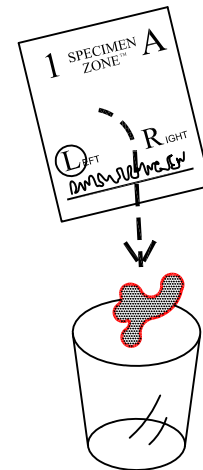

SANDEL
the original exclusive sponsor of
AORN
Patient Safety First program.

In the healthcare industry, the color orange is a trademark of Sandel Medical Industries, LLC.

SANDEL LLC
Leaders in Healthcare Safety

Sandel Medical Industries, LLC
19736 Dearborn Street, Chatsworth, CA 91311
1-818-534-2500 www.sandelmedical.com