


DIRECTIONS FOR USE: PETITE MARKER™

Pat. # US D538,851


Reorder No. 1011-NS1 Petite Skin Marker (NON-STERILE)
Reorder No. 1002-NS Petite Permanent Marker (NON-STERILE)

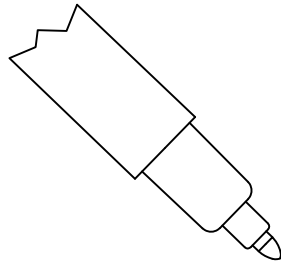

Skin Marker Ink.

- 1. Remove purple cap for Skin marker use.**
- 2. Replace cap after use.**

1011-NS1 Petite Skin Marker

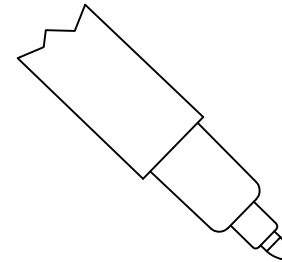

Permanent Ink.

- 1. Remove black cap for Permanent marker use.**
- 2. Replace cap after use.**

1002-NS Petite Permanent Marker

Important!

1. Permanent Ink is not intended to be used on the skin.
2. For single patient use only.
3. Dispose of after use.
4. Skin surface must be clean and dried before marking the skin.
5. Alcohol or alcohol-based solutions will remove marks.



Sandel Medical Industries, LLC
19736 Dearborn Street, Chatsworth, CA 91311 USA
818-534-2500 www.sandelmedical.com