



Directions For Use:

CORRECT SITE™ TATTOO

1. Skin should be clean & free of oils & makeup.
2. Remove clear, protective top sheet.
3. Press tattoo firmly onto clean, dry skin with design facing down.
4. Hold wet cloth against back of tattoo, press down and make sure to wet it thoroughly.
5. Wait 30 seconds (don't hurry). Peel off paper backing.

To Remove: Use rubbing alcohol

WARNING: Do not apply over surgical site / Do not cut through tattoo.

Ingredients: Acrylic Multipolymer, Alkyd Polyester Resin, Phenolic Modified Gloss Varnish, Petroleum Isoparaffinic Hydrocarbon, Petrolatum, Cobalt Tallate Drier, Black Iron Oxide, FD & C Blue #1 Aluminum Lake, FD & C Yellow #5 Aluminum Lake, D & C Red #7.

260038 Rev.A

Sandel Medical Industries, LLC

19736 Dearborn Street, Chatsworth, CA 91311 ▪ *tel* 818-534-2500 *fax* 818-534-2510 www.sandelmedical.com

SANDEL is the original exclusive sponsor of AORN Patient Safety First program